HALF ROLLY 890.02

CE

Regolamento (UE)
2017/745

IT

WWW.KONG.IT

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Declares under its sole responsibility that the device:

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V08050199 - Stretchers for the transfer of patients – others

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Stretcher for transporting patients during rescue operations

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CONTENTS

1 - SYMBOLS AND ASSISTANCE	4
1.1 Symbols	4
1.2 Assistance	4
2 - GENERAL INFORMATION	5
3 - TECHNICAL FEATURES	6
3.1 Nomenclature and materials for the parts	6
3.2 Dimensions	8
3.3 Capacity	8
3.4 Optional components and spare parts	9
4 - SPECIFIC INFORMATION	11
4.1 Intended use	11
4.2 Preparation	11
4.3 Setup	12
4.4 Immobilising the patient	18
4.5 Pairing modalities with compatible accessories	20
4.6 Transporting the patient	21
5 - MAINTENANCE AND REPAIRS	22
5.1 General	22
5.2 Maintenance	22
5.3 Repairs	22
6 - STORAGE	23
7 - INSPECTIONS AND SERVICING	24
7.1 Checks	24
7.2 Inspections	24
7.3 Servicing	24
8 - PRODUCT LIFE AND GUARANTEE	25
8.1 Product lifespan	25
8.2 Disposal	25
8.3 Guarantee	25
8.4 Legal obligations	25
9 - MARKINGS	26
9.1 Marking	26
9.2 Symbols	26
10 - REGISTRATION DOCUMENTS	27
10.1 Maintenance and repairs register	27
10.2 Recording inspections and servicing	28
10.2 EU Declaration of conformity	29
11 - LIST OF APPLIED AND REFERENCE STANDARDS	30
11.1 Applied standards	30
11.2 Reference standards	31

CHAPTER

1

SYMBOLS AND SUPPORT

1.1 SYMBOLS

For the sake of comfort and clarity while reading this manual, below are the symbols used for handling important warnings for a proper and safe use of the device.

REQUIREMENT FOR PROPER USE

It identifies the presence of information for proper use of the device.

INFORMATION REQUIREMENT

It identifies the presence of useful and general information which guides the reader towards a conscious use of the device and/or performance of actions.

It identifies that the Medical Device is manufactured, designed and produced in compliance with the provisions of the General Safety and Performance Requirements of Regulation (EU) 2017/745 (Class I medical device, in compliance with classification rule 1 as indicated in the attachment VIII).

1.2 SUPPORT

For information please contact Kong Customer Support Service by:

- telephone +39 0341 630506
- fax +39 0341 641550
- email: safetycare@kong.it

or write to KONG S.p.A. – Via XXV Aprile, 4 – 23804 Monte Marenzo LC - ITALY.

To facilitate support operations, please always communicate or state the serial number (SN) indicated on the label fixed to the Medical Device.

CHAPTER 2

2 GENERAL INFORMATION

The information provided by the manufacturer (hereinafter referred to as information) must be read and clearly understood by the user prior to using the Medical Device (MD hereafter). The information regards the description of the features, performance, assembly, disassembly, maintenance, preservation, disinfection, etc. of the device. Even though the information offers tips on use, this information shall not be considered as a user manual under actual conditions of use.

WARNINGS AND LIMITATIONS OF USE:

- this device shall be strictly used by people who are physically fit, trained (instructed and taught) to use the device and with specific experience regarding moving the patient or, during training activities, by people under direct supervision of the trainers/ supervisors who guarantee the safety thereof,
- do not use the device before fully reading and understanding this user manual,
- strictly follow the manufacturer's information, improper use of the device is hazardous,
- modifying and/or repairing the device is strictly forbidden,
- all checks described in chapter 7 must be carried out prior to and after using the device. In case of any doubt on the efficiency of the device, the user must replace it immediately,
- non-compliant use, deformations, falls, wear, chemical contamination, exposing textile/plastic components/devices to temperature below -30°C or above

- +50°C and metal components/devices to temperatures exceeding 100°C are some examples of causes that can reduce, limit and end the life of the device,
- Prior to any rescue operation, be keen not to exceed the capacity indicated in paragraph 3.3,
- in order to reduce risks of exposure to / transmission of infectious diseases, clean and disinfect the device as indicated in chapter 5,
- improper use of the patient immobilisation systems can jeopardise the safety of the patient,
- always check the compatibility of the devices used alongside the device by consulting the relative manufacturer's information,
- use of spare parts or optional components different from the ones indicated in paragraph 3.4 can be hazardous,
- do not expose the device to sources of heat and at contact with chemical substances. Reduce exposure to direct sunlight as much as possible. At low temperatures and in the presence of humidity, formation of ice could reduce flexibility and increase cutting and abrasion-related risks on textile and synthetic devices,
- Report to the manufacturer and the competent authority of the member state where the user and / or patient is established any serious incident occurring in relation to the MD.

All our devices are tested/ checked piece by piece in compliance with the procedures laid down by the Quality System certified in accordance with the UNI EN ISO 9001 standard. Laboratory tests, testing, information and standards do not always reproduce the practical result. Thus, the results obtained under the actual conditions of use of the device in the natural environment may differ, even considerably at times. The best information lies in the continuous practical use under the supervision of skilled/ expert/qualified people.

Warning: not suitable for use in an ATEX environment (Directive 94/9/CE)

CHAPTER 3

TECHNICAL FEATURES

3.1 NOMENCLATURE AND MATERIALS FOR THE PARTS

- A - Support surface in high-density polyethylene,
- B - Slings in polyester,
- C - Webbing loops in polyester,
- D - Handles in polyester,
- E - Sling in polyester,
- F - Buckles with hook in aluminium alloy,
- G - Automatic buckles,
- H - Head immobilizer HRP,
- I - Head immobilizer chin strap,
- L - Thigh straps,
- M - Static rope,
- N - Sling with ratchet,
- O - Transport bag.

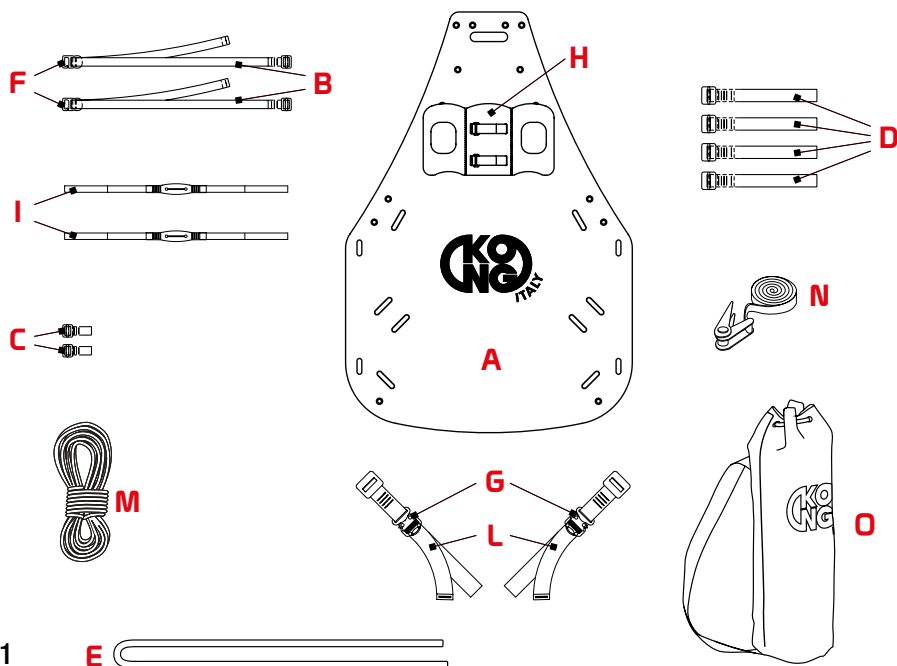
Fig.1

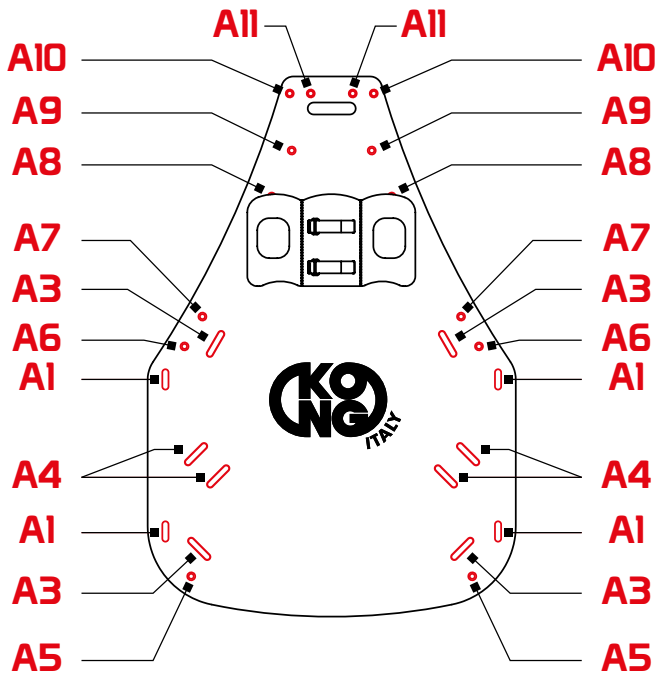
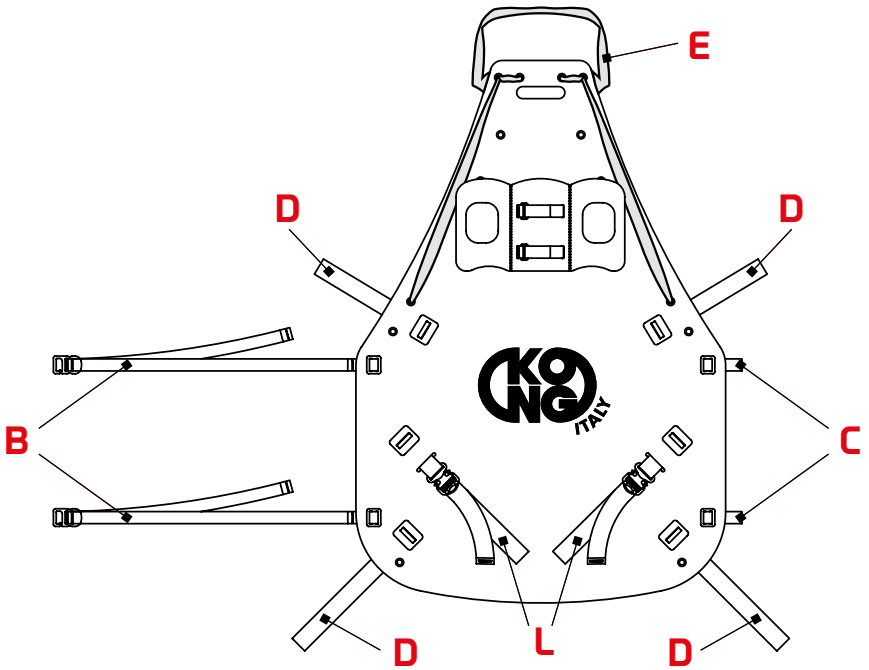
Fig.2

3.2 DIMENSIONS

Length:	135 cm
Width:	90 cm
Rolled in the bag:	ø 30 x 110 cm
Overall weight of the parts (fig. 1):	6.6 kg

3.3 CAPACITY

The table below defines the capacities of the “HALF ROLLY” rescue sheet, based on the method of use deemed most appropriate by the rescue operator, the outcome of the laboratory tests and recommended safety coefficients.

How to use	Capacity
Transport by the handles (D) (fig. 3)	150 kg
Dragging by the sling (E) (fig. 4)	
Lifting and lowering with devices with rescue devices (M) (fig. 5-6)	

Before any recovery operation, make sure that the weight does not exceed the capacity defined capacity!

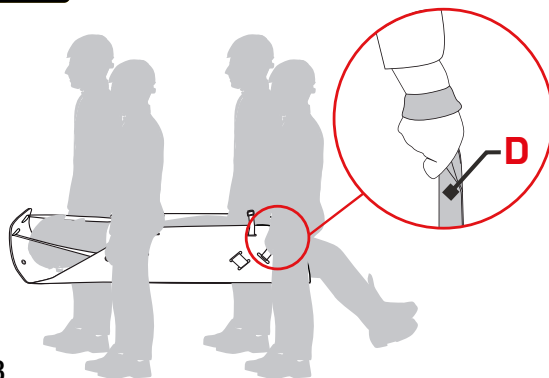
Fig.3

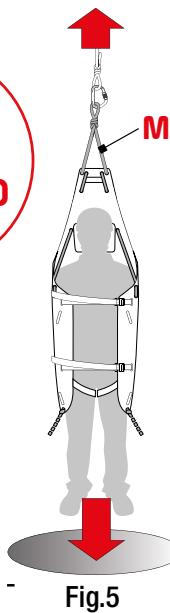
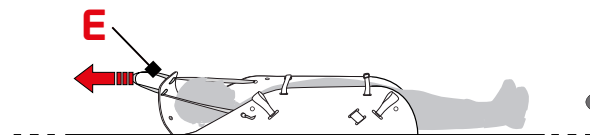
Fig.5

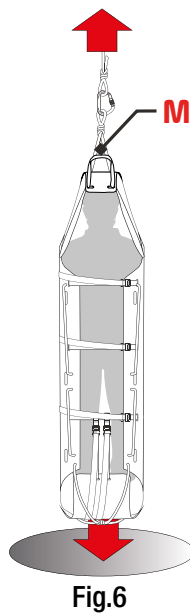
Fig.6

3.4 OPTIONAL COMPONENTS AND SPARE PARTS

3.4.1 Optional components

The “**HALF ROLLY**” rescue sheet is complete with all optional components for the intended purposes indicated in this manual.

3.4.2 Spare parts

- B - Slings in polyester,
- C - Webbing loops in polyester,
- D - Handles in polyester,
- E - Sling in polyester,
- H - Head immobilizer HRP
- I - Head immobilizer chin strap,
- L - Thigh straps,
- M - Static rope,
- N - Sling with ratchet,
- O - Transport bag.

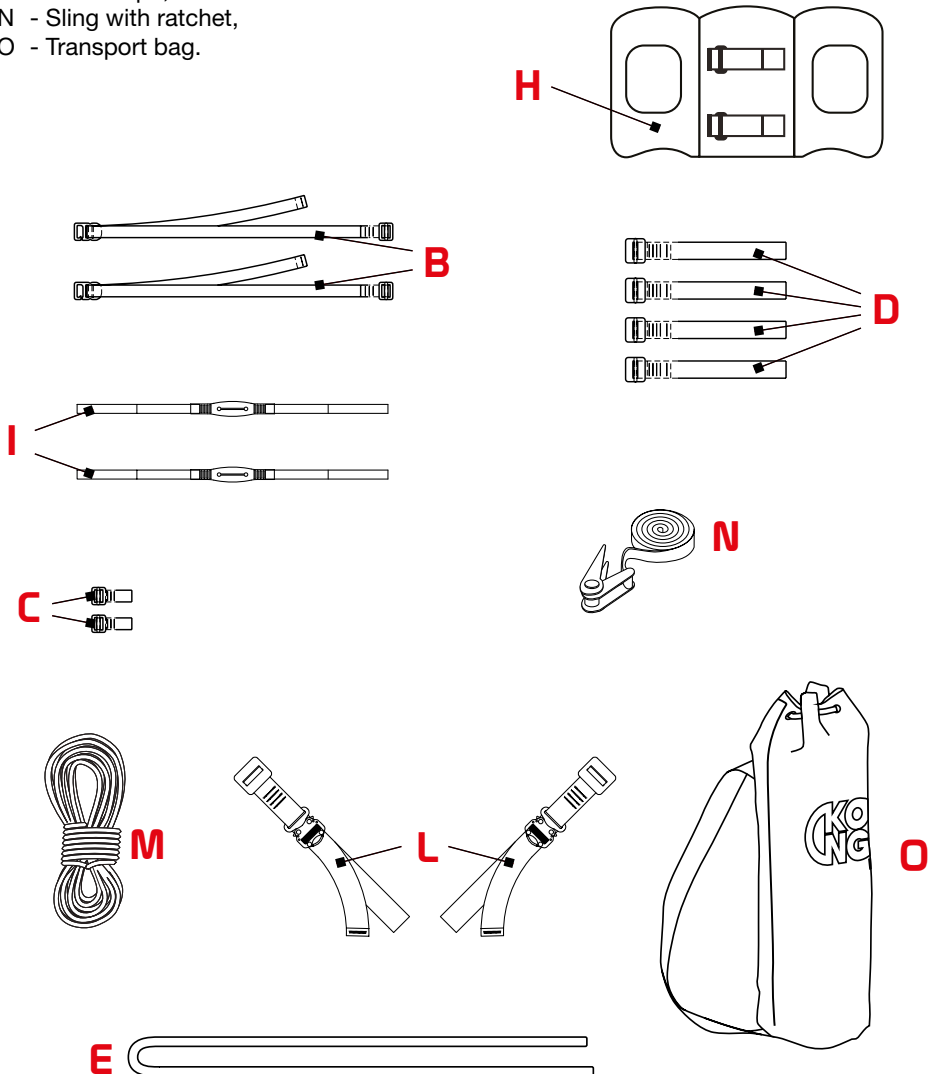
Fig.7

CHAPTER 4

SPECIFIC INFORMATION

4.1 INTENDED USE

The **“HALF ROLLY”** rescue sheet is a medical device suitable for extracting the patient from particularly narrow and tight spaces.

Decisions concerning handling and immobilization of the patient, as well as the duration, the procedures to be applied, and any possible combination with other devices must be taken only by expert and trained personnel.

The use in combination with the 890.01 ROLLY sheet improves the lifting / lowering and transport conditions of the patient.

4.2 PREPARATION

- Remove **“HALF ROLLY”** rescue sheet from the transport bag (O) and remove the ratchet sling (N),
- Distend the support surface (A) and flatten it, rolling the ends over in the opposite direction and folding the central section (fig. 8)

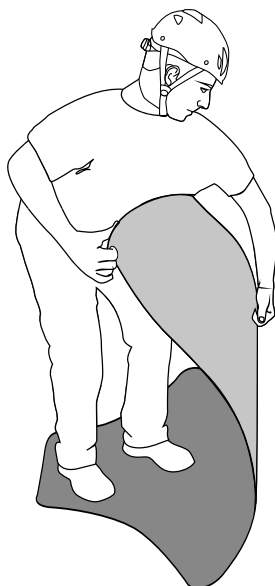
Fig.8

4.3 SETUP

4.3.1 Setting up for immobilising the patient

Prepare the following setup to immobilise the patient:

- Insert the sling (E) in the A7 holes in the support surface (fig. 9).
- Insert the slings (B) in the A1 eyelets in the support surface (fig. 10).
- Insert the webbing loops (C) in the A2 eyelets in the support surface (fig. 11).
- Insert the thigh straps (L) in the A4 eyelets in the support surface and release the automatic buckles (G) (fig. 12).

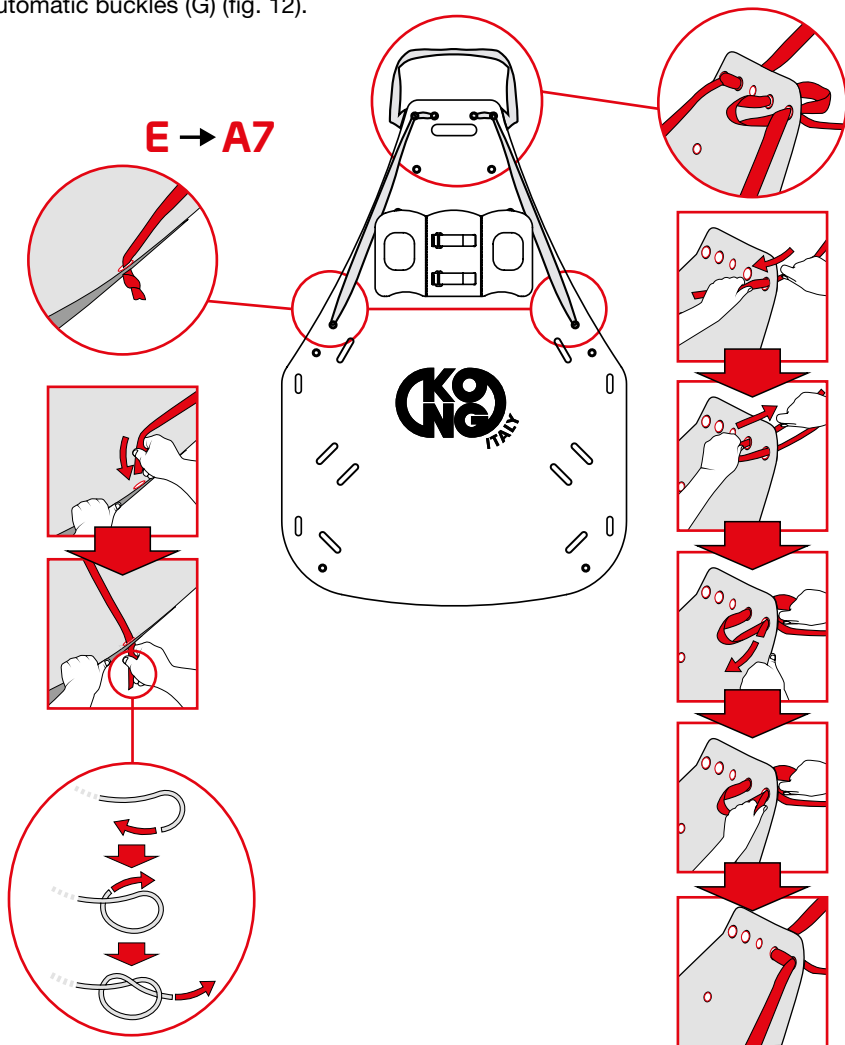
Fig.9

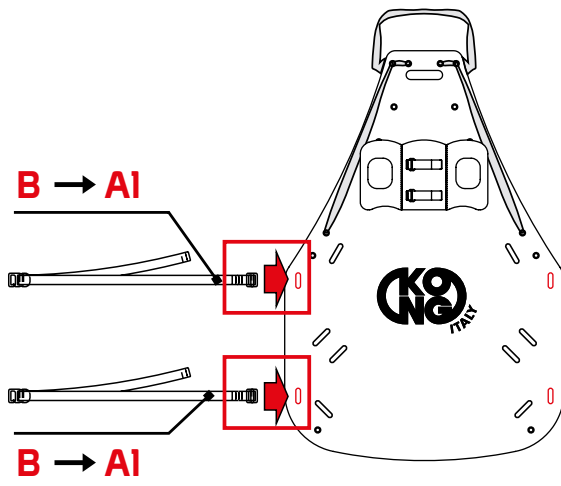
Fig.10

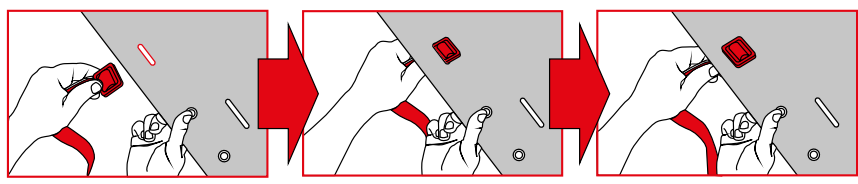
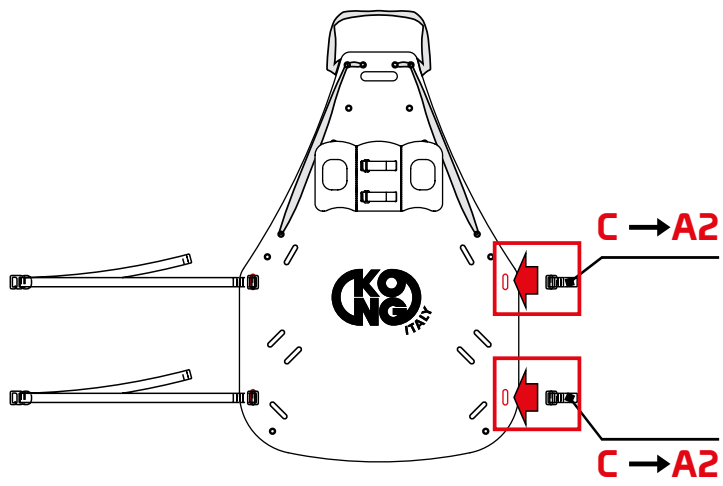
Fig.11

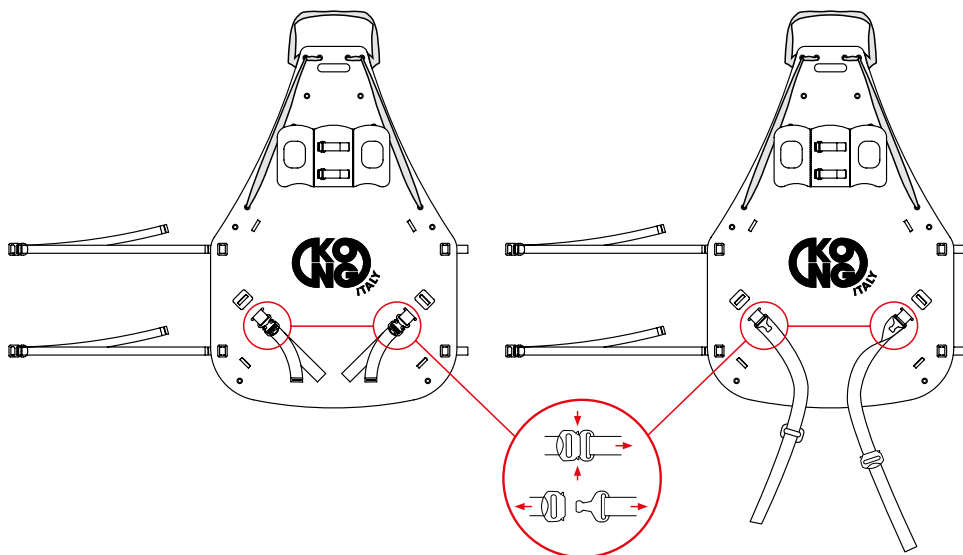
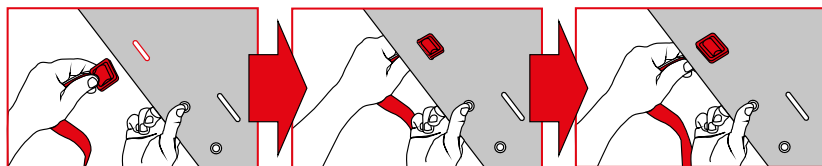
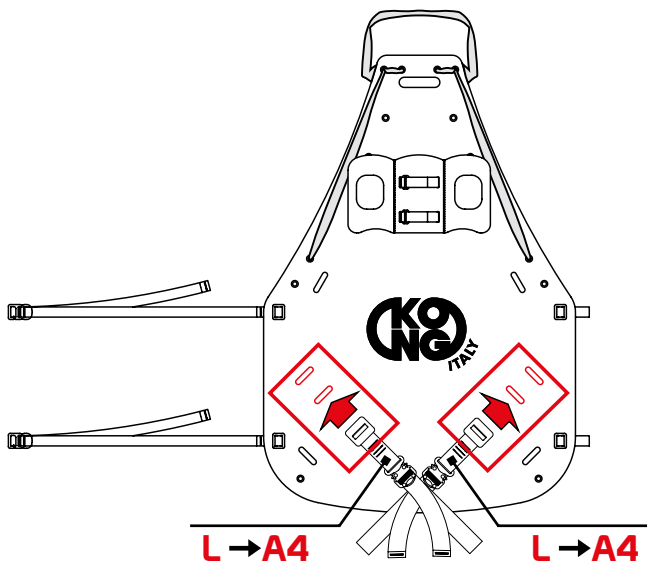
Fig.12

4.3.2 Setting up for moving the patient

In relation to the dragging, transport and lifting/dropping procedures that the rescue worker considers suitable, set the stretcher up with at least one of the following parts:

- Dragging: tension the sling (E) to lift the end part of the support surface (A) enough to contain the head of the patient and immobilise it (fig. 13).

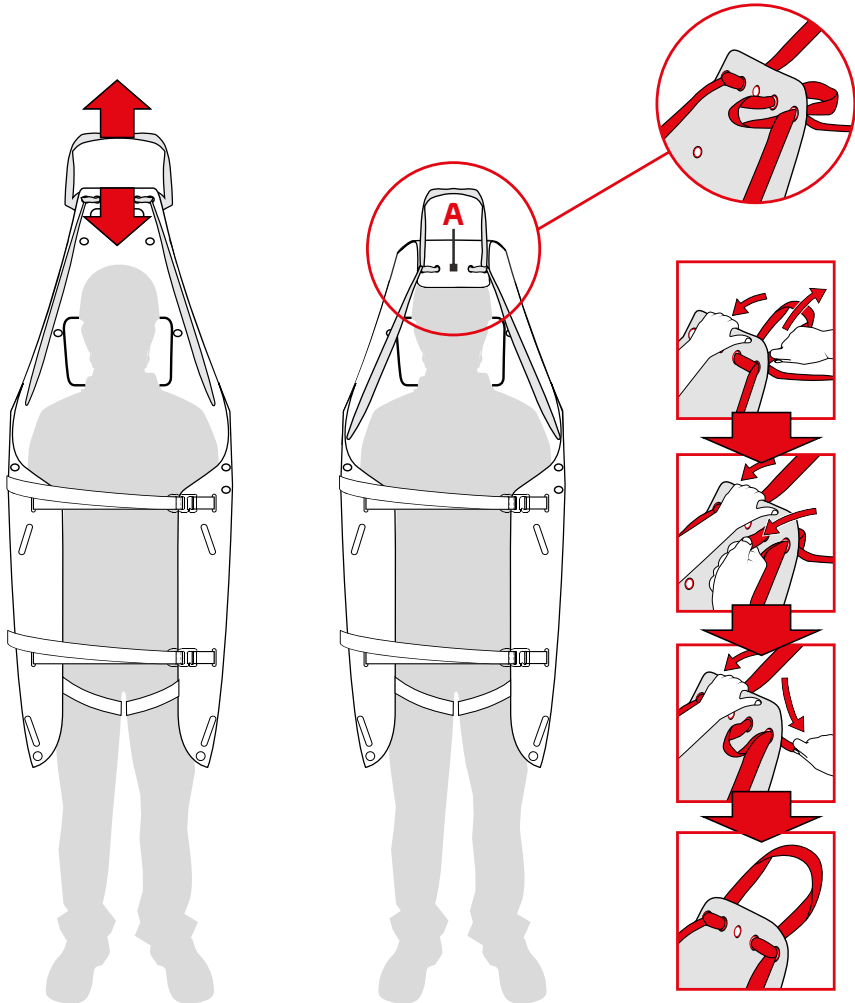
Fig.13

b) Manual transport: insert the side handles (D) in the A3 eyelets in the support surface (fig. 14).

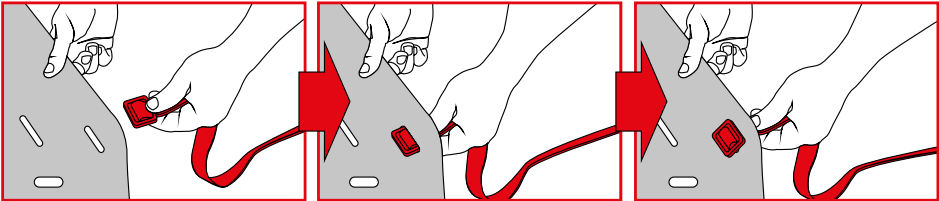
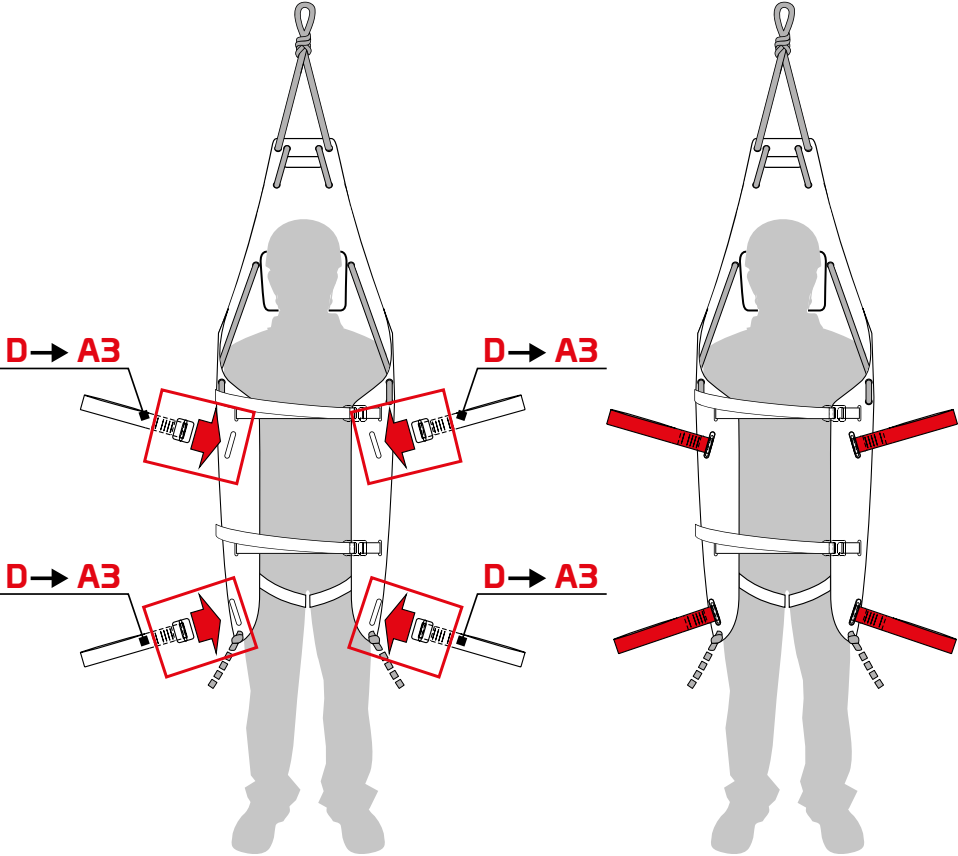
Fig.14

c) Vertical lifting/lowering: insert the rope (M) in the holes from A5 to A11 in the support surface (fig. 15).

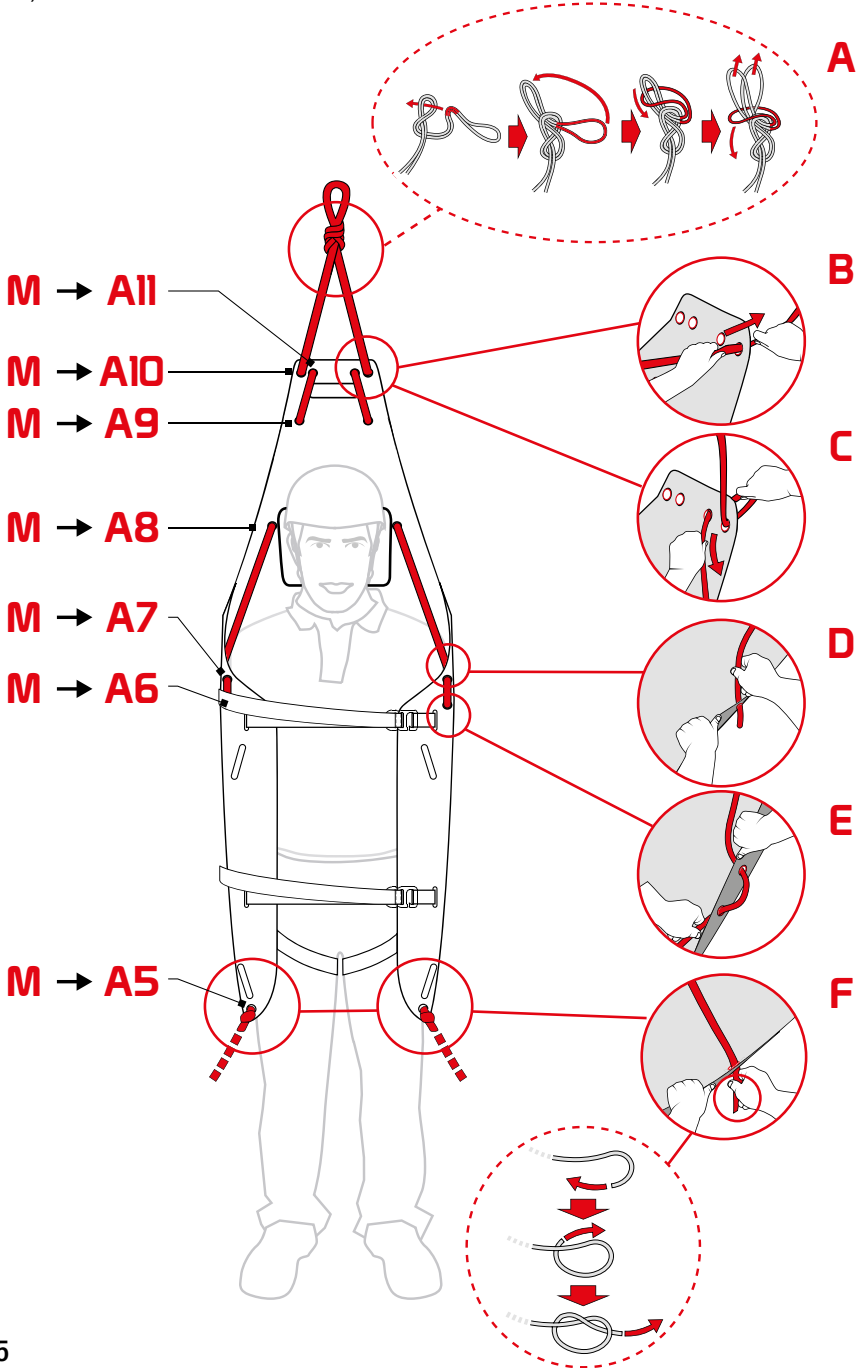
Fig.15

4.4 IMMOBILISING THE PATIENT

After having positioned the patient on the support surface (A):

- Release the automatic buckles (G) and tension the thigh straps (L) - (fig. 16),
- Insert the buckles with hook (F) in the corresponding webbing loops (C) and tension the slings (B) to lift the side parts of the support surface (A) enough to contain the patient (fig. 17).

Avoid prolonged contact of the patient's skin with the fabrics of the device.

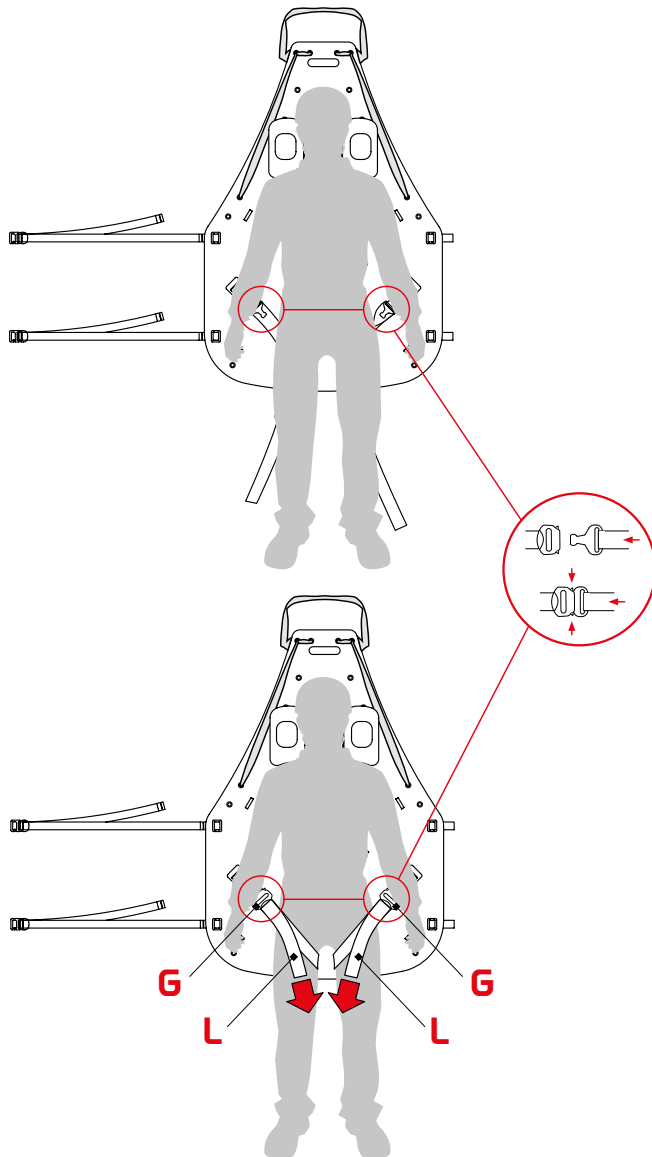
Fig.16

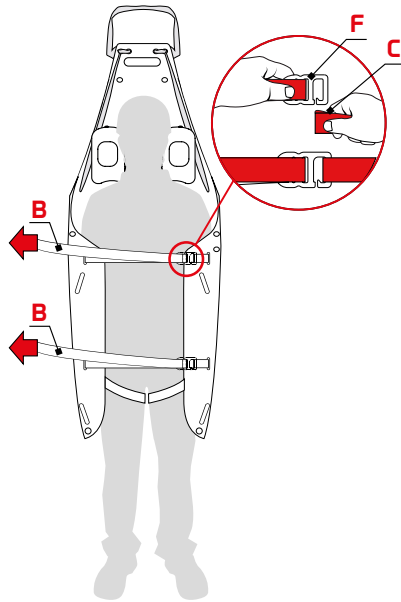
Fig.17

If the rescue worker thinks it is necessary, it is possible to use the head immobilizer HRP (H) to immobilise the patient's head, with the following procedures:

- a) Fasten the chin strap (I) to the head immobilizer HRP (H), using the velcro (fig. 18),
- b) Wind the chin strap velcro (I) around the rope (M) or the sling (E) and make it adhere to itself (fig. 18).

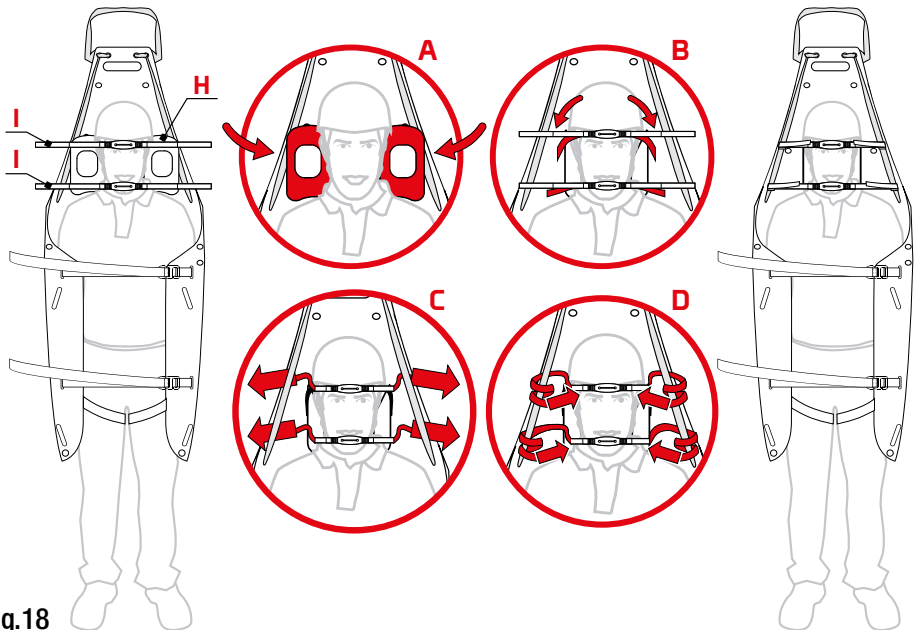
Fig.18

4.5 PAIRING MODALITIES WITH ROLLY SHEET 890.01

To improve the quality of lifting, lowering, and transport of the patient assured at the “HALF ROLLY”, pair the accessory device “ROLLY” with the following modalities:

- Distend the “**ROLLY**” sheet, set up in relation to the planned operating mode,
- Position the “**HALF ROLLY**” with the immobilised patient on top, and make sure that the holes are aligned (fig. 19), and then insert the rope (M) - fig. 20.

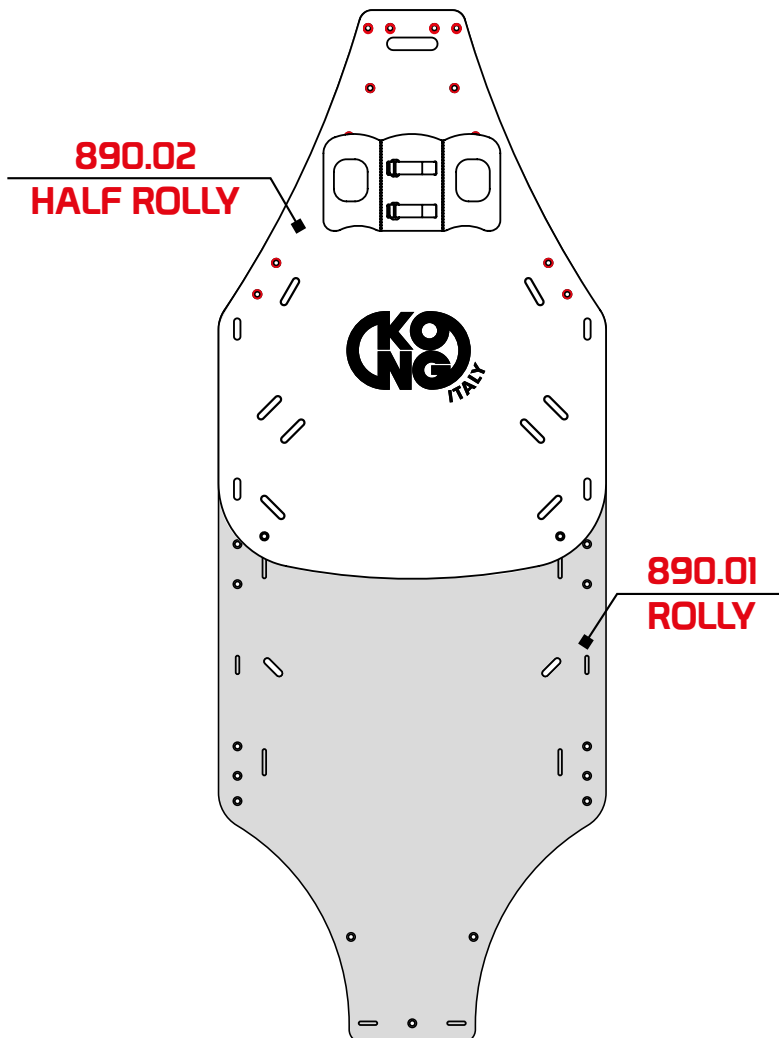
Fig.19

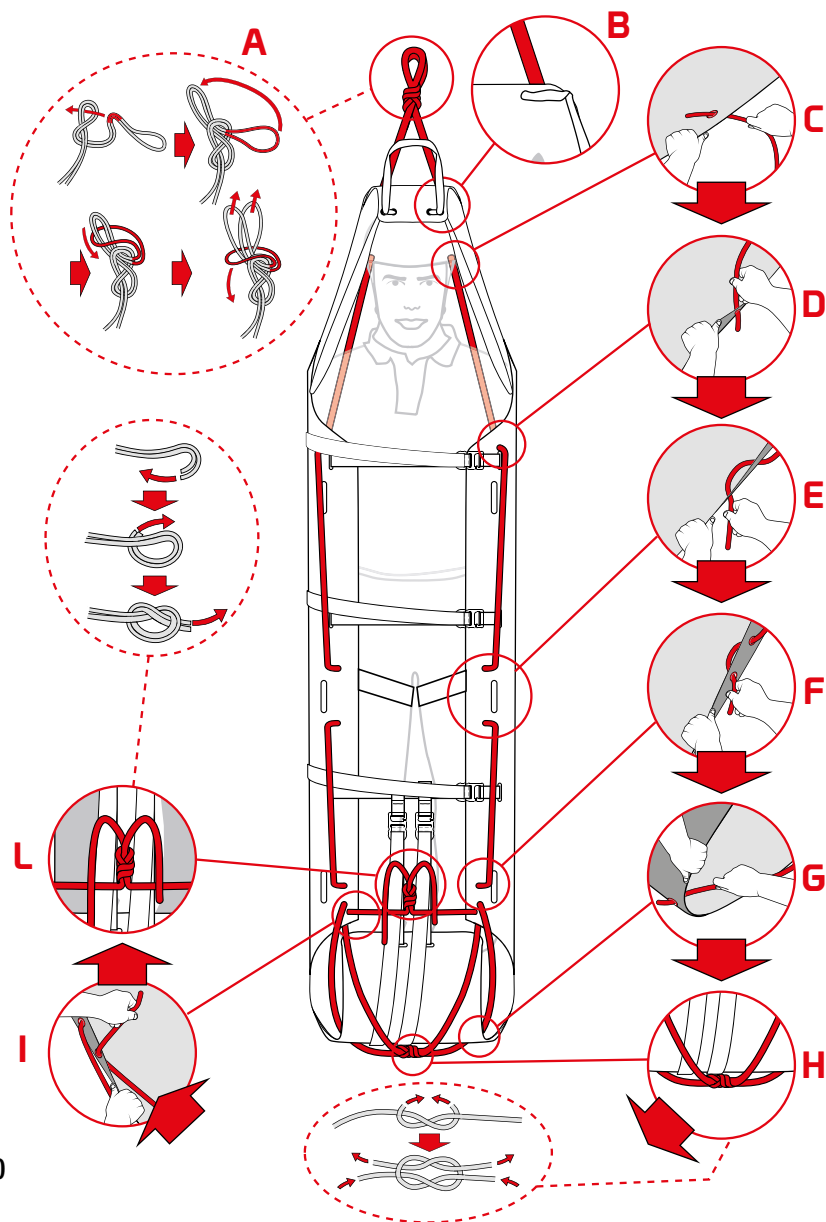
Fig.20

4.6 TRANSPORTING THE PATIENT

The “**HALF ROLLY**” rescue sheet is suitable for lifting and transporting the patient using the side handles (fig. 3), dragging them using the sling (E) (fig. 4), and lifting/lowering them vertically using the static rope (fig. 5 e 6).

CHAPTER 5

MAINTENANCE AND REPAIRS

5.1 GENERAL

The “**HALF ROLLY**” rescue sheet is made of materials that are highly resistant to wear and to external agents. Despite this, the conditions of use make it necessary to perform maintenance and, in particular cases, even repairs.

Maintenance and repairs must be recorded on a designated register, an example of which is shown in chapter 10.

5.2 MAINTENANCE

The maintenance operations that must be carried out by the user are, in order:

- a) cleaning: after every use, wash with warm (max. 40 °C) potable water, possibly adding a neutral detergent (e.g. Marseille soap). Rinse and leave to dry in a shaded place, away from direct sources of heat,
- b) disinfection, when deemed necessary or after prolonged inactivity (more than 21 days), preceded by cleaning: wipe with a clean cloth, soaked in a water solution of sodium hypochlorite (bleach) with a 0,1% concentration (1000 ppm). When blood or other bodily fluids are present, the recommended concentration of sodium hypochlorite is 0,5% (5000ppm).

5.3 REPAIRS

Repairs must be carried out by the manufacturer only. Users are allowed only to replace the parts mentioned in paragraph 3.4.2 with new and original parts.

CHAPTER 6

6 STORAGE

After cleaning, disinfection and drying, store the device in a dry (40-90% relative humidity), fresh (temperature 5-40°C) and dark (avoid U.V. radiation) place, which is also chemically neutral (absolutely avoid salty and/or acid environments), away from sharp edges, sources of heat, dampness, corrosive substances or other possible detrimental conditions.

Do not store this device when wet!

CHAPTER 7

CHECKS INSPECTIONS AND SERVICING

7.1 CHECKS

Before and after every use, check the device and ensure that:

- it is appropriate for the use it was intended for,
- it has not been subjected to mechanical deformations and it neither has cracks nor shows signs of wear,
- the textile parts do not have cuts, burns, chemical products residues, excessive hair, wear, in particular check areas at contact with the metal components,
- markings, including labels, are readable.

7.2 INSPECTIONS

The device must be inspected at least once a year, starting from the date of first use, by expert personnel trained and approved by KONG S.p.A. The date of the first use and the outcome of the inspection must be recorded in the Inspections and Servicing Register, regarding which an example is indicated in chapter 10.

7.3 REVISIONS

The device must be serviced by KONG S.p.A., or by expert personnel trained and approved by KONG S.p.A. if:

- any malfunctions are detected,
- the result of the pre and post use check or of the inspections is negative.

The outcome of the servicing must be recorded in the Inspections and Servicing Register, regarding which an example is indicated in chapter 10.

The device subject of servicing has a one-year warranty from the date of servicing.

CHAPTER 8

DURATION AND WARRANTY

8.1 DURATION

Carefully read the “Warnings and limitations of use” paragraph in chapter 2. The duration of this device depends on the outcome of the at least one-yearly inspection (paragraph 7.2) and possible servicing.

Important: any devices that have not passed the pre-use, post-use and periodic inspection must be eliminated and made unusable.

8.2 DISPOSAL

Follow the rules in force in the country of use and the waste disposal procedures of the reference hospital facility for proper disposal.

8.3 WARRANTY

The manufacturer guarantees the compliance of the device with the regulations in force at the time of production. Warranty covering defects is limited to defects regarding raw materials and manufacturing defects. The warranty does not cover normal wear, oxidation, damage caused by non-compliant use and/or in competition, by improper maintenance, transportation, preservation or storage. Warranty will be immediately deemed null and void should the device be modified or tampered with. Validity corresponds to the legal validity of the country where the device was sold, as from the date of sale, by the manufacturer. No claims shall be raised against the manufacturer once the aforementioned period has expired. Any repair or replacement request under warranty must be accompanied by a proof of purchase. Should the defect be recognised, the manufacturer undertakes to repair or, replace or refund the device at his discretion. The manufacturer shall not be held liable beyond the invoice price of the device in any case whatsoever.


8.4 LAW OBLIGATIONS

Professional and free time activities are often regulated by special national laws which can impose limits and/or obligations concerning the use of these devices. The user has the obligation to know and apply such laws which could provide for limits different from those indicated in this information.

CHAPTER 9

LABELLING AND SYMBOLS

9.1 MEDICAL DEVICE LABEL

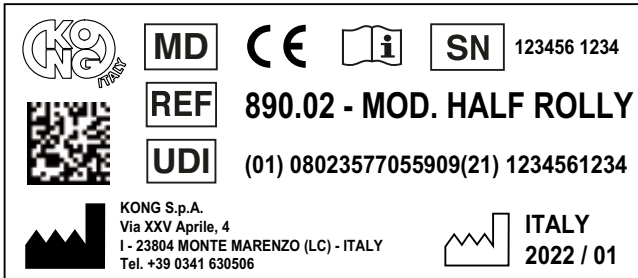Any changes to the positioning of the symbols does not change the content thereof.

9.2 SYMBOLS

Identification and traceability

	Medical Device
	Product identification code
	Unique Device Identifier
	Manufacturer's identification
	Unique serial number
	Country, year and month of production.

Data matrix UDI
 Readable with the Apps:
 iGepir - BarValid - Barcode Scanner

Information and safety

	Consult the user manual
	Compliant with Regulation (EU) 2017/745

CHAPTER 10

REGISTRATION DOCUMENTS

10.1 INSPECTIONS AND SERVICING REGISTER

RESCUE SHEET - Class I Medical Device				
REF.	890.02	FORM	HALF ROLLY	
UDI-DI	08023577055909	SN		
Date of first use				
INSPECTION AND SERVICING				
DATE	I/S	Description	Outcome	Manager

10.2 MAINTENANCE AND REPAIR REGISTER

RESCUE SHEET - Class I Medical Device				
REF.	890.02	MOD.	HALF ROLLY	
UDI-DI	08023577055909	SN		
Date of first use				
MAINTENANCE AND REPAIRS				
DATE	M/R	Description	Outcome	Supervisor

10.2 EU DECLARATION OF CONFORMITY

The EU Declaration of Conformity is available for download at www.kong.it

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11.1 APPLIED STANDARDS

- EN 597-1: 2015
Furniture - Assessment of the ignitability of mattresses and upholstered bed bases - Part 1: Ignition source smouldering cigarette
- EN 1865-1: 2010 + A1: 2015
Patient handling equipment used in road ambulances - Part 1: General stretcher systems and patient handling equipment
- EN 62366-1: 2015 + A1: 2020
Medical devices - Application of usability engineering to medical devices
- EN ISO 846:2019
Plastics - Evaluation of the action of microorganisms
- EN ISO 10993-1: 2020
Biological evaluation of medical devices - Part 1: Evaluation and testing within a risk management process
- EN ISO 10993-5: 2009
Biological evaluation of medical devices - Part 5: Tests for in vitro cytotoxicity
- EN ISO 10993-23: 2021
Biological evaluation of medical devices - Part 23: Tests for irritation
- EN ISO 14971: 2019
Medical devices - Application of risk management to medical devices
- EN ISO 15223-1: 2021
Medical devices — Symbols to be used with information to be supplied by the manufacturer — Part 1: General requirements
- EN ISO 20417: 2021
Medical devices - Information to be supplied by the manufacturer

11.2 STANDARDS USED AS REFERENCE

- EN ISO 13485: 2016
Medical devices - Quality management systems - Requirements for regulatory purposes
- EN 13718-1: 2014 + A1: 2020
Medical vehicles and their equipment - Air ambulances - Part 1: Requirements for medical devices used in air ambulances

KONG s.p.a

Via XXV Aprile, 4 – (industrial area)

I-23804 MONTE MARENZIO (LC) – ITALY

Tel +39 0341630506 – Fax +39 0341841550

www.kong.it